INDICATIONS: For non-surgical or post-operative support and immobilization of the injured shoulder where glenohumeral positioning is key.

CAUTION: This device is not intended for reuse on a second patient or for extended use beyond 180 days.

CAUTION: Activity modification and proper rehabilitation are an essential part of a safe treatment program. If any additional pain or other symptoms occur while using this device, discontinue use and seek medical attention.

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CAUTION: This device is not intended to prevent injury, or reduce or eliminate the risk of re-injury.

CAUTION: This device has been developed for the indications listed above, and may be damaged by unusual activities such as sports, falls, or other accidents. To avoid the risk of accident and possible injury to the patient, discontinue use if such an incident has occurred.

CAUTION: Do not leave this device in a car on a hot day, or in any other place where the temperature may exceed 150F (65 C.) for any extended period of time.

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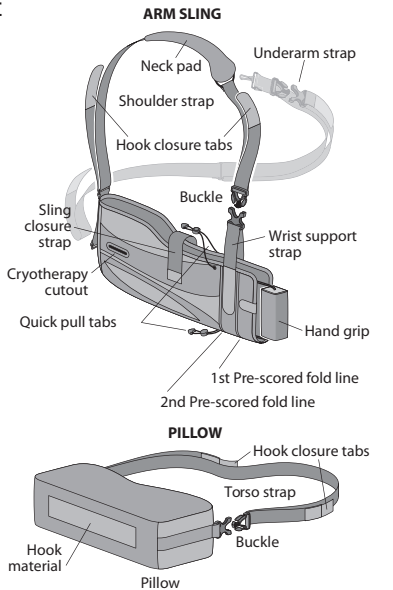
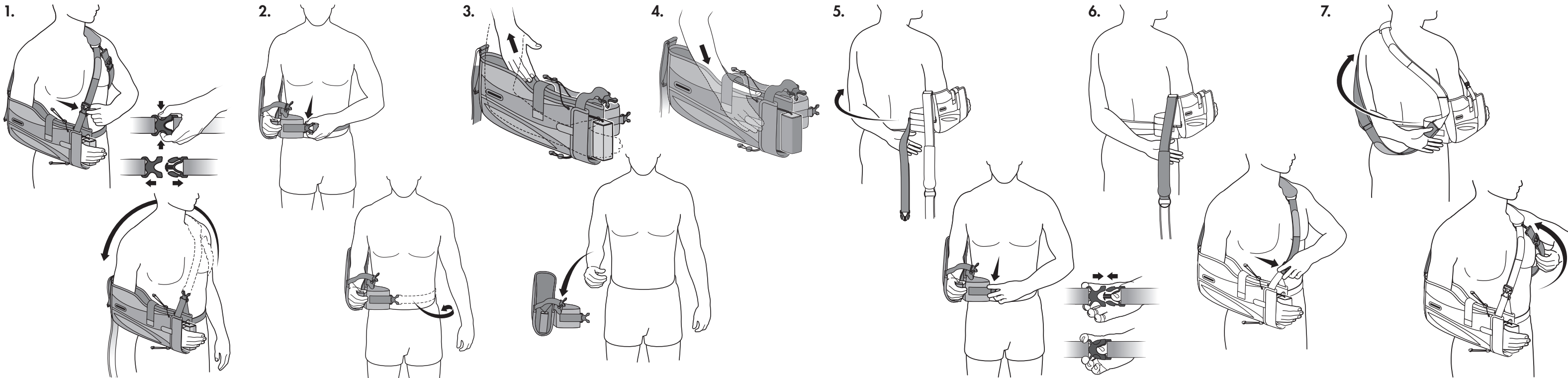
TO REMOVE THE ARC 2.0 WITH PILLOW

Example: Right Shoulder

1. Release the buckle of the shoulder strap, and let it slide off the back shoulder. If the underarm strap is being used, release the buckle leaving it clipped to the ring inside the neck pad.
2. Release the buckle of the pillow torso strap, and let it fall behind the back. Leave the pillow attached to the arm sling.
3. Remove the sling and pillow as one unit from the affected arm. Leave the sling and pillow attached to each other for re-application later.

TO RE-APPLY THE ARC 2.0 WITH PILLOW

Example: Right Shoulder

4. Slip affected arm into the arm sling. The arm sling and pillow should still be attached to one another.
5. With unaffected arm, reach behind your back and pull the pillow torso strap around to the front and attach the buckle at the front.
6. With unaffected arm, reach behind your back and pull the shoulder strap around to the front and clip the buckle at the wrist support strap of the arm sling.

TO RE-APPLY WITH THE UNDERARM STRAP ATTACHED

7. Slip the unaffected arm between the shoulder strap and remaining strap. Insert the remaining buckle on the underarm strap to secure.

NOTICE TO USERS

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.

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© 2023 Breg, Inc. All rights reserved.