X2K® ADJUSTABLE HINGE, COUNTERFORCE PLUS and X2K-OA FITTING INSTRUCTIONS

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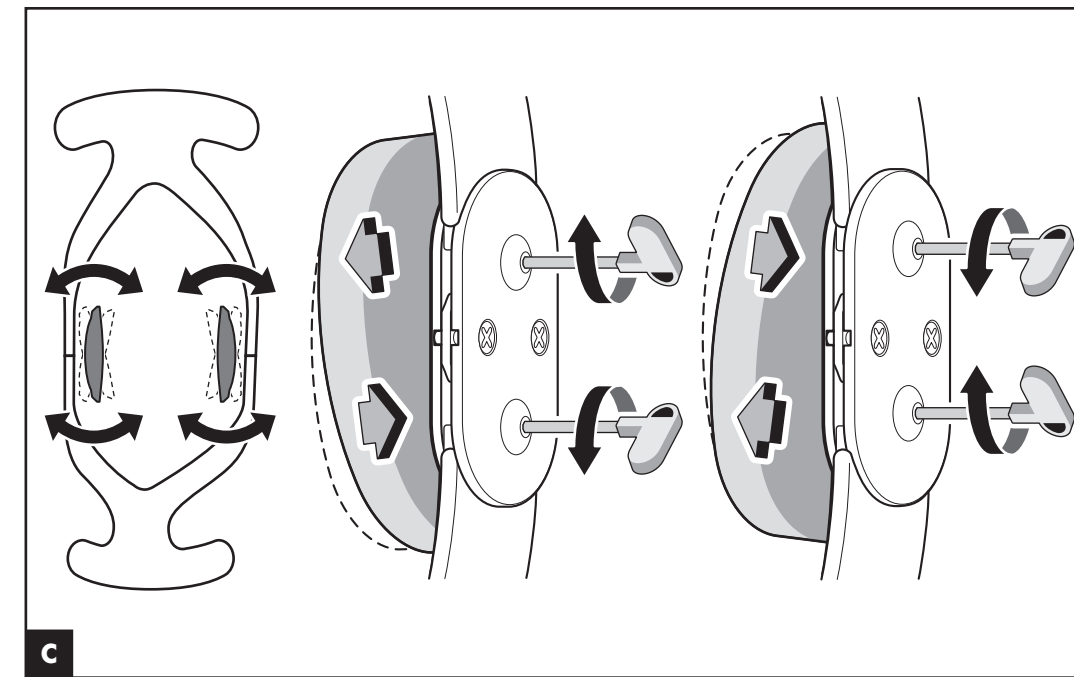
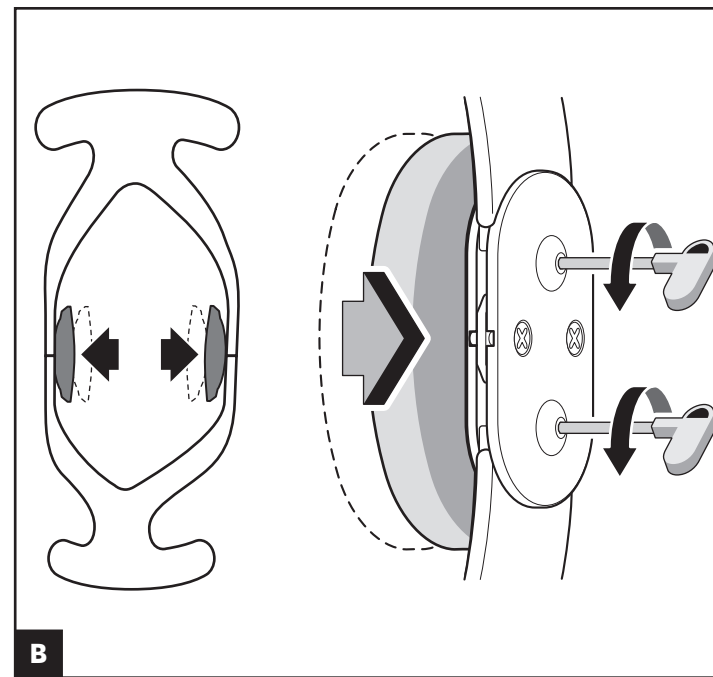
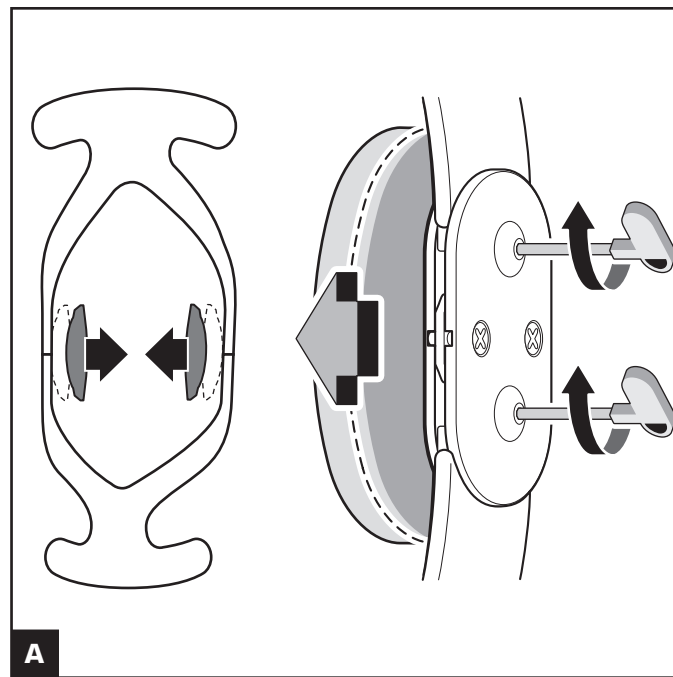
WARNINGS

- CAREFULLY READ FITTING INSTRUCTIONS AND WARNINGS PRIOR TO USE.
- FOR SINGLE PATIENT USE ONLY.
- IF YOU EXPERIENCE INCREASED PAIN, SWELLING, SKIN IRRITATION, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.
- THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO ESSENTIAL PARTS OF A SAFE TREATMENT PROGRAM. CONSULT YOUR LICENSED HEALTH CARE PROFESSIONAL SHOULD YOU HAVE ANY QUESTIONS REGARDING THE SAFE APPLICATION OF THIS PRODUCT RELATED TO YOUR MEDICAL HISTORY.

CAUTION: CARE AND CLEANING OF THIS PRODUCT IS ESSENTIAL TO ITS CONTINUING STRENGTH AND PERFORMANCE. FOLLOW REMOVAL AND CLEANING PROCESS DESCRIBED BELOW.

CAUTION: WHEN USING BRACES BILATERALLY, MODIFY ACTIVITY AS NECESSARY TO KEEP MEDIAL HINGES FROM CONTACTING EACH OTHER.

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Adjustable Hinge Instructions

- A ADJUSTMENT FOR LOOSE FITTING PADS**
Use tool provided to turn upper and lower adjustment screws on sides of medial and lateral hinges clockwise. This will move condyle pads inward, tighter against knee. Adjust for comfortable, consistent fit.
- B ADJUSTMENT FOR TIGHT FITTING PADS**
Turn upper and lower adjustment screws on sides of medial and lateral hinges counter clockwise. This will move condyle pads outward, away from knee.
- C ADJUSTMENT IF TOO LOOSE OR TIGHT AT UPPER OR LOWER EDGE OF PADS**
Condyle pads may be tilted to fit contour of the knee. Upper or lower edge of pads may be adjusted to fit more tightly or loosely by turning upper or lower adjustment screws. Turn screws counter clockwise to move pad edge outward, away from knee. Turn screws clockwise to move pad edge inward, tighter against knee.

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© 2023 Breg, Inc.