

CrossRunner™ & CrossRunner Short

Pull-On

Instructions For Use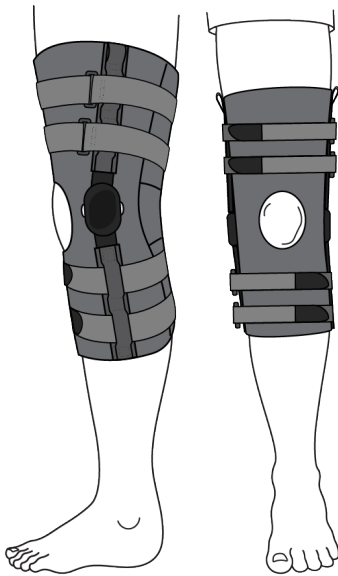

CrossRunner

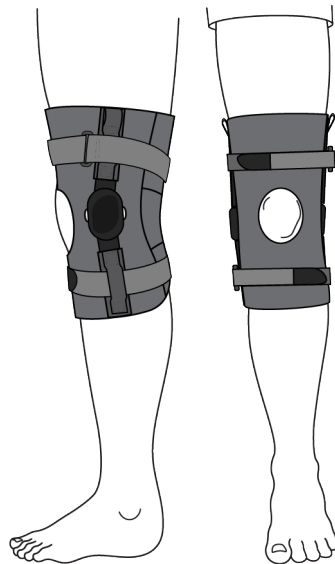
CrossRunner Short

2382 Faraday Avenue, Suite 300
Carlsbad, CA 92008 USA
Toll Free Tel: 1-800-321-0607
Local: +1-760-795-5440
Fax: +1-760-795-5295
www.breg.com

©2023 Breg, Inc. All rights reserved.
CrossRunner is a trademark of Breg, Inc.

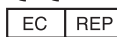E/U authorized representative
MDSS GmbH
Schiffgraben 41
D-30175 Hannover
Germany

CrossRunner & CrossRunner Short Knee Brace

Table of Contents

Fitting Instructions:

Instrucciones de ajuste Instructions de pose Anleitungen zur korrekten Anlegung Pasinstructies Instrukcije zakladanja Istruzioni per l'adattamento Kiinnitysohjeet Justeringsanvisningar Bruksanvisning for tilpasning Tilpasningsvejledning Instruções de ajuste	3
---	---

Hinge Instructions:


Instrucciones para la articulación Instructions relatives aux charnières Scharnieranleitungen Scharnierinstructies Instrukcja obsługi zawiasów Istruzioni per lo snodo Saranoiden ohjeet Gångjärnsanvisningar Bruksanvisning for hengsel Hængselsvejledning Instruções sobre as dobradiças	
--	--

<i>English</i>	4
<i>Español</i>	6
<i>Français</i>	8
<i>Deutsch</i>	10
<i>Nederlands</i>	12
<i>Polski</i>	14
<i>Italiano</i>	16
<i>Suomi</i>	18
<i>Svenska</i>	20
<i>Norsk</i>	22
<i>Dansk</i>	24
<i>Português</i>	26


ROM Stop Guide - Scale 1:1

Guía de topes del ROM - Escala 1:1 Guide de butée d'amplitude articulaire - Échelle 1:1 ROM Stopp-Führung - Maßstab 1:1 Handleiding bewegingsbereikaanslag - schaal 1:1 Instrukcja obsługi ograniczników ROM Stop - Skala 1:1 Guida per il fermo ROM - Scala 1:1 Liikealueen rajoittimen ohje - suhde 1:1 Guide för rörelseomfångsstopp - skala 1:1 BO-stopper vejledning - skala 1:1 ROM-stop-guide - Skala 1:1 Guia da trava de ADM - Escala 1:1	28
--	----


1.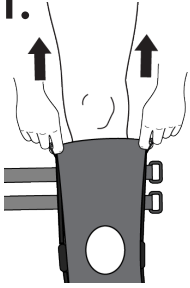


2.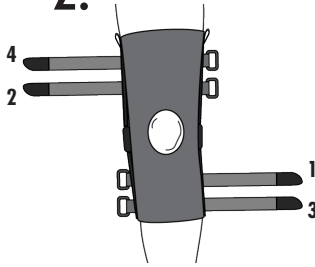


3.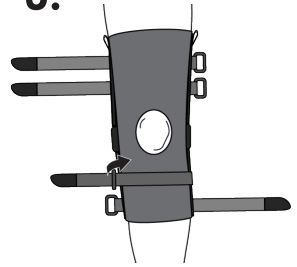


4.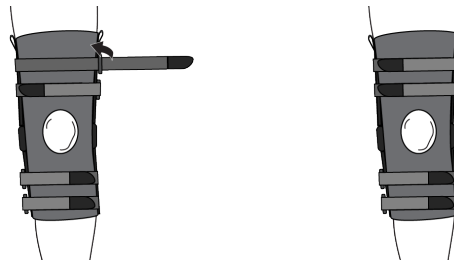


A.

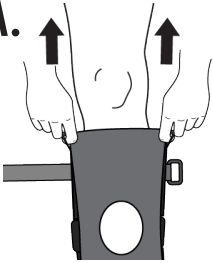
B.

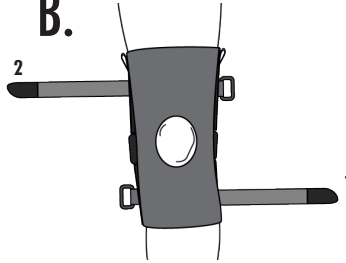C.

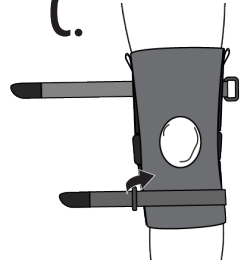D.

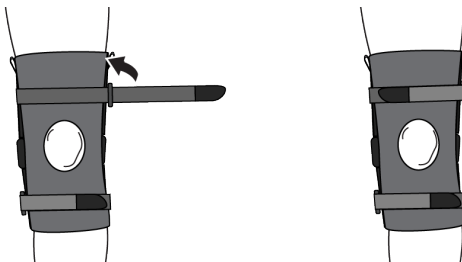1. Pull up brace
Tire la rodillera hacia arriba
Tirez l'attelle vers le haut
Hochziehbare Stütze
Trek de brace omhoog
Naciğnaç ortezę
Tirare il tutore verso l'alto
Vedä tuki ylös
Dra upp ortosen
Dra opp støtten
Træk ortesen op
Vista a Joelheira

3. Loop strap 1 through ring and secure
Pase la correa 1 por la anilla y ajústela
Passez la boucle 1 par l'anneau et attachez-la
Führen Sie den Gurt 1 durch den Ring und sichern Sie ihn
Voer band 1 door de ring en maak hem vast
Przełożyć pasek 1 przez klamrę i zapiąć
Far passare la cinghia 1 attraverso l'anello e fissarla
Kierrä hihna 1 renkaan läpi ja kiinnitä
Dra rem 1 genom ringen och fäst den
Før rem 1 gjennom ringen og fest den
Træk strap 1 gennem ringen, og fastgør den
Passe a tira 1 pelo anel e prenda-a

2. Align hinge with knee
Alinee la articulación con la rodilla
Alignez la charnière sur le genou
Richten Sie das Scharnier mit dem Knie aus
Lijn het scharnier uit met de knie
Umieścić zawias na wysokości kolana
Allineare lo snodo con il ginocchio
Kohdistá sarana polven kanssa
Justera gångjärnet till knät
Tilpass støtten til kneet
Ret hængslet ind efter knæet
Alinhe a dobradiça com o joelho

4. Loop and secure remaining straps
Pase y ajuste el resto de correas
Bouclez et attachez les autres sangles
Führen Sie die restlichen Gurte durch und sichern Sie sie
Voer de resterende banden door de ringen en maak ze vast
Przełożyć przez klamry i zapiąć pozostałe paski
Far passare attraverso gli anelli e fissare le cinghie rimanenti
Kierrä ja kiinnitä loput hihnat
Dra igenom och fäst de återstående remmarna
Før de andre remmene gjennom på samme måte og fest dem
Træk og fastgør de resterende stropper
Prenda as tiras restantes com um laço

A. Pull up brace
Tire la rodillera hacia arriba
Tirez l'attelle vers le haut
Hochziehbare Stütze
Trek de brace omhoog
Naciğnaç ortezę
Tirare il tutore verso l'alto
Vedä tuki ylös
Dra upp ortosen
Dra opp støtten
Træk ortesen op
Vista a Joelheira

B. Align hinge with knee
Alinee la articulación con la rodilla
Alignez la charnière sur le genou
Richten Sie das Scharnier mit dem Knie aus
Lijn het scharnier uit met de knie
Umieścić zawias na wysokości kolana
Allineare lo snodo con il ginocchio
Kohdistá sarana polven kanssa
Justera gångjärnet till knät
Tilpass støtten til kneet
Ret hængslet ind efter knæet
Alinhe a dobradiça com o joelho

C. Loop strap 1 through ring and secure
Pase la correa 1 por la anilla y ajústela
Passez la boucle 1 par l'anneau et attachez-la
Führen Sie den Gurt 1 durch den Ring und sichern Sie ihn
Voer band 1 door de ring en maak hem vast
Przełożyć pasek 1 przez klamrę i zapiąć
Far passare la cinghia 1 attraverso l'anello e fissarla
Kierrä hihna 1 renkaan läpi ja kiinnitä
Dra rem 1 genom ringen och fäst den
Før rem 1 gjennom ringen og fest den
Træk strap 1 gennem ringen, og fastgør den
Passe a tira 1 pelo anel e prenda-a

D. Loop strap 2 and secure
Pase la correa 2 y ajústela
Passez la sangle 2 dans son anneau et attachez-la
Gurt 2 anlegen und sichern
Voer band 2 door de ring en maak hem vast
Przełożyć pasek 2 przez klamrę i zapiąć
Far passare la cinghia 2 attraverso il rispettivo anello e fissarla
Kierrä hihna 2 ja kiinnitä
Dra igenom rem 2 och fäst
Dra rem 2 gjennom og fest den
Træk strap 2, og fastgør den
Passe a tira 2 e prenda-a

Warnings

⚠ WARNING: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, SKIN IRRITATION, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.

⚠ WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM. CONSULT YOUR LICENSED HEALTH CARE PROFESSIONAL REGARDING SAFE AND APPROPRIATE ACTIVITY LEVEL WHILE WEARING THIS DEVICE.

CAUTION: UNITED STATES FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.

CAUTION: FOR SINGLE PATIENT USE ONLY.

CAUTION: CAREFULLY READ FITTING INSTRUCTIONS AND WARNINGS PRIOR TO USE. TO ENSURE PROPER PERFORMANCE OF THE BRACE, FOLLOW ALL INSTRUCTIONS. FAILURE TO PROPERLY POSITION THE BRACE AND FASTEN EACH STRAP WILL COMPROMISE PERFORMANCE AND COMFORT.

CAUTION: USE IN HIGH-IMPACT ACTIVITIES MAY CAUSE PREMATURE WEAR OF COMPONENTS.

CAUTION: CARE AND CLEANING OF THIS PRODUCT IS ESSENTIAL TO ITS CONTINUING STRENGTH AND PERFORMANCE. FOLLOW REMOVAL AND CLEANING PROCESS DESCRIBED IN INSTRUCTIONS.

Care and Cleaning

- ATTACH STRAP ENDS TO STRAP PRIOR TO CLEANING.
- AFTER EXPOSURE TO SALT WATER OR DIRT, THOROUGHLY RINSE THE BRACE WITH FRESH WATER AND ALLOW TO AIR DRY.
- A DRY LUBRICANT, SUCH AS TEFLON SPRAY, MAY BE USED ON THE HINGE OCCASIONALLY. DO NOT USE WET LUBRICANTS.
- THE HINGE ASSEMBLIES MAY BE REMOVED TO ALLOW EASIER WASHING OF THE SLEEVE AND STRAPS.
- THE SIDE BARS MAY BE BENT TO ACCOMMODATE ANATOMY. BEND IN EQUAL AMOUNTS ABOVE AND BELOW THE HINGE.
- PERIODICALLY INSPECT THE SLEEVE, STRAPS AND HINGES FOR EXCESSIVE WEAR. BE SURE STOPS ARE IN PLACE.

Range of Motion (ROM) Stop


CrossRunner Adjustment Instructions — Polycentric hinge with Stops:

Dual function extension and flexion ROM Stops

- Every Stop may be used to control Flexion or Extension
- 7 Unique Stops provide 14 angle set positions:
 - 80 Flexion / 10 Extension (Pre-Installed)*
 - 45 Flexion / 45 Extension
 - 50 Flexion / 40 Extension
 - 60 Flexion / 30 Extension (Quantity 2)
 - 70 Flexion / 20 Extension
 - 75 Flexion / 15 Extension
 - 90 Flexion / 0 Extension (Quantity 2)



ROM Stop Removal Key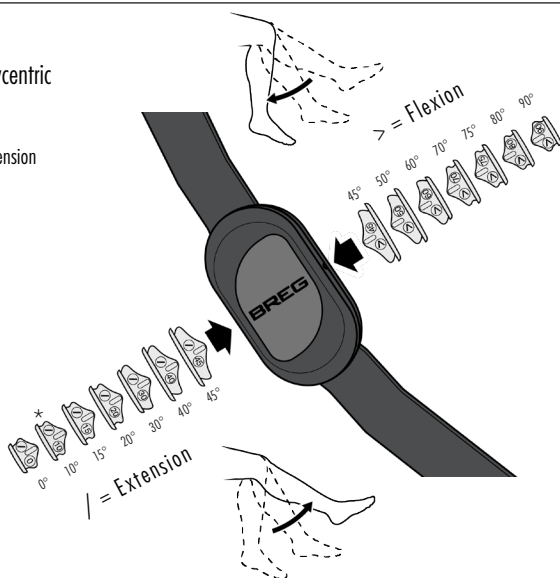


ROM Stop Removal

⚠ Warning: Do not point ROM Stops at anyone's head or face during removal. The Removal Key or a small screwdriver can remove the Stops from the hinge.


- Grip the hinge as shown then follow either removal method below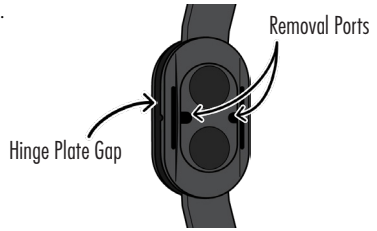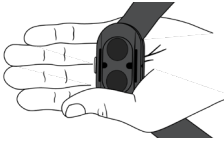

- Removal port method
 - Insert the ROM Stop Removal Key into the port
 - Leverage the Key to partially eject ROM Stop
 - Remove the Stop manually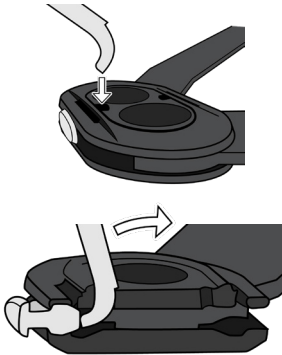

- Hinge plate gap method
 - Insert the ROM Stop Removal Key into the hinge plate gap by the ROM Stop
 - Rotate the Key to eject the ROM Stop

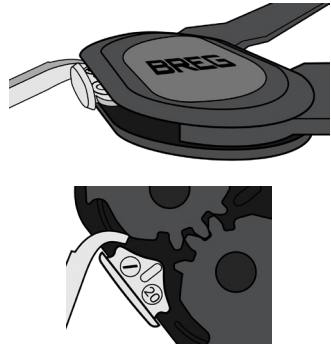
Insertion of the ROM Stops

- Select the ROM Stop
 - Extension – Anterior side
 - Flexion – Posterior side
- Press firmly to snap in ROM Stop

Extension

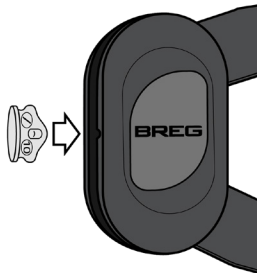Flexion

IF ANY SERIOUS INCIDENT HAS OCCURRED IN RELATION TO THE DEVICE, REPORT TO BREG AND THE COMPETENT AUTHORITY OF THE MEMBER STATE IN WHICH THE USER AND/OR PATIENT IS ESTABLISHED.

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