



# Achilles Boot Rise Kit

INSTRUCTIONS FOR USE AW-CP020127 Rev M 0224

TOTAL LEG SUPPORT SYSTEM WITH INFINITE R.O.M. & SHORT APPLICATION TIME



THE BREG ACHILLES RISE KIT CONTAINS:

1 - GRAY INNERSOLE W/ CUTOUT PIECE  
2 - 18" STRAPS  
1 - INSTRUCTION SHEET

1 - SET OF (4) 10° ACHILLES WEDGES  
4 - 2" X 2" DOUBLE SIDED HOOK-PIECES

This device is offered for sale by or on the order of a physician or other qualified medical professional. This device is not intended for reuse on a second patient or for extended use beyond 180 days. This device is generally not intended for patients in excess of 250 lbs (114 kilos). This device is not intended to prevent injuries or reduce or eliminate the risk of reinjury.

**Indications:** Indicated for immediate application following injury to or reconstruction of the Achilles tendon.

**Contraindications:** Contraindicated for unstable fractures or for fractures of the proximal tibia or fibula.

**WARNING:** Do not leave this device in the trunk of a car on a hot day or in any other place where the temperature may exceed 150° F (65° C) for any extended period of time. **If any additional pain or symptoms occur while using this device, seek medical attention.**



© 2024 Breg, Inc. All Rights Reserved  
2382 Faraday Avenue, Suite 300  
Carlsbad, CA 92008 U.S.A.  
Toll Free Tel: 1-800-321-0607  
Local: +1-760-795-5440  
Fax: +1-760-795-5295  
www.Breg.com



EC REP

E/U authorized representative  
MDSS GmbH  
Schiffgraben 41  
D-30175 Hannover  
Germany



# Achilles Boot Rise Kit

## INSTRUCTIONS FOR USE



**1. REMOVE BOOT INNER-SOLE** Peel out the inner-sole material inside the foot area of the boot.




**2. APPLY KIT INNERSOLE** Apply the kit innersole in the Breg Boot leaving 1/8" (.32cm) space on each end of the innersole. Save the cutout piece for step 8.



**3. REMOVE FOREFOOT STRAPS** Disconnect the pile to hook fastener and remove the strap from the D-Ring.

**NOTE:** The old 2" X 2" double-sided hook pieces will be used to attach the forefoot pad to the straps.

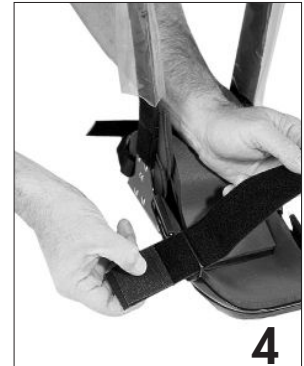

**4. NEW FOREFOOT STRAPS**

Apply the kit straps through the D-Rings and stick back on themselves with the 2" X 2" double-sided hook pieces.

**NOTE:** Cut off the excess strap after applying the boot to the patient.



**5. REMOVE WEDGES** To change or remove the wedges, pry the upper wedge assembly up and save.



**6.** Peel out each 10° wedge as necessary for patient progress.



**7.** Replace the upper wedge assembly.



**8. REPLACE CUTOUT**

After all wedges are removed, the saved cut out piece can be replaced for further rehab.

### Notice To Users

If any serious incident has occurred in relation to the device, report to Breg and the competent authority of the Member State in which the user and/or patient is established.