

Part number: AW-1.03438

Description: PC WAVE ICE BAG

Revision: A 01/19

4.125 in

----- Water Fill Line -----

Ice Bag Preparation


- 1 Fill 4 bags with water to Water Fill Line (approximately 2 cups).
- 2 Freeze bags.

Unit Setup

- 1 Add water to fill line in cooler.
- 2 Place 4 frozen bags in corners of cooler.

Tip:
While using the Breg Ice Bags, cubed ice may also be added to the container up to the ice fill line to optimize the colder setting.

**FIRST ADD WATER
SECOND ADD BREG ICE BAGS**

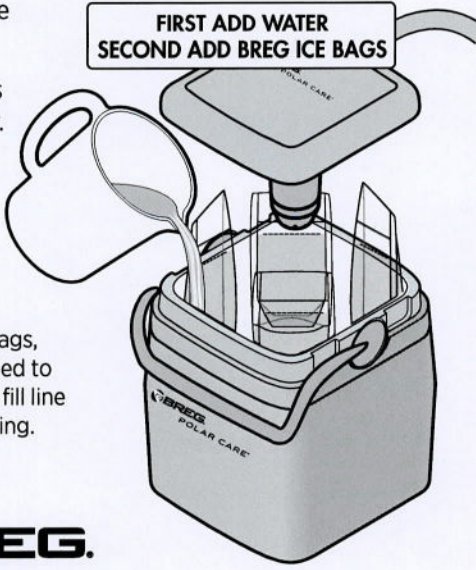

BREG.

©2019 Breg, Inc. All rights reserved.
AW-1.03438 Rev A 01/19

6.5 in

PINK DOES NOT PRINT